

Thank you for choosing ANA.

For your information, airport staff and cabin attendants cannot explain or operate your oxygen cylinder.

Instructions for Oxygen Cylinders

Rentals

1. The medical oxygen cylinders will be handed to you in cabin.
2. Please return the cylinders to our staff in cabin.

Attachment

- Instruction(this paper)
- Mask, Cannula

How to use

- ① Put on a Mask or Cannula.
- ② Check the dial of flow regulator is set to "0".
- ③ Set the flow of oxygen according to your physician's instruction by turning the dial.
- ④ Turn the shut-off valve slowly to open.
(Caution: Just turn. Do not remove the pin.)


Note : Oxygen flow can be set from 0.25 to 6.0 liters/min.(International: ~5.0litters/min)

Storage space

Please place the cylinder under the seat in front.



⚠Caution: Turn the valve slowly.

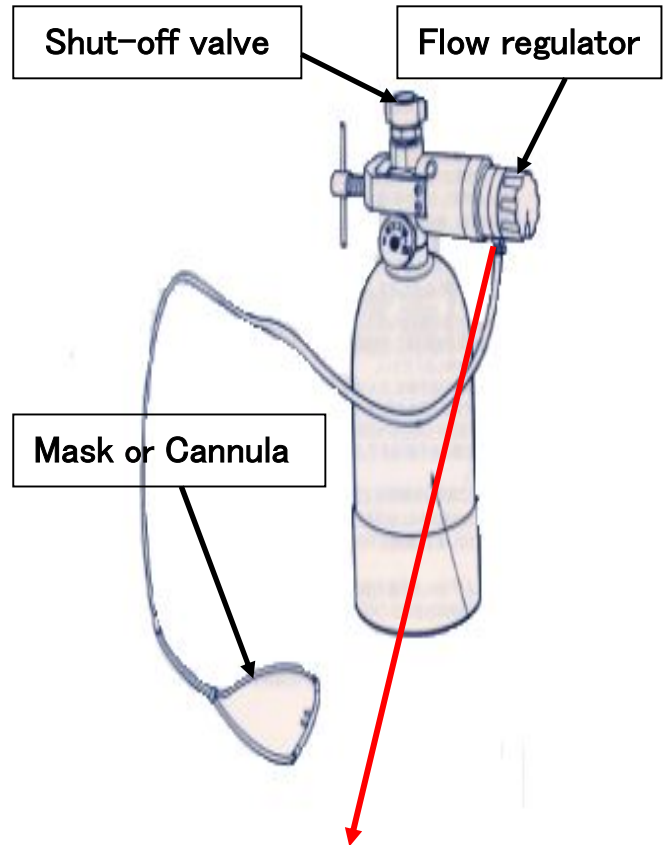

There are two types of flow regulator.



Domestic flight



International flight