






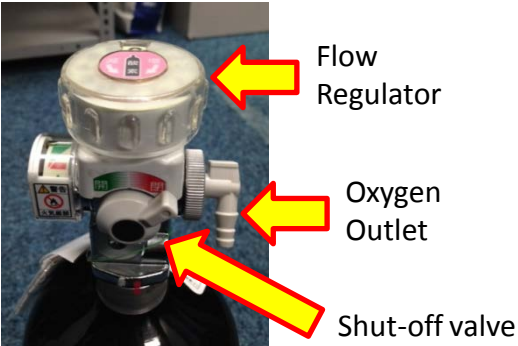



Thank you for boarding.

For your information, airport staff and cabin attendants cannot explain or operate an oxygen cylinder.



Instructions for Oxygen Cylinders 【Domestic】

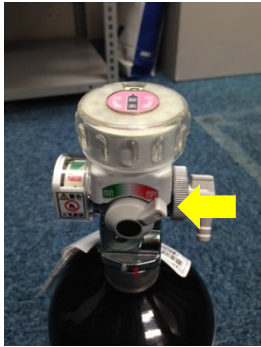Functions	Attachments
 <p>Flow Regulator</p> <p>Oxygen Outlet</p> <p>Shut-off valve</p>	 <p>Mask</p>  <p>Cannula</p>
<p>How to use</p> <p>① Install a Mask or Cannula into the Oxygen Outlet.</p> 	<p>② Check the dial of flow regulator is set to "0".</p> 
<p>③ Turn the shut-off valve to open (GREEN) completely.</p> 	<p>④ Set the flow of oxygen according to your physician's instruction by turning the dial.</p> 
<p>Method of the end</p> <p>① Turn the shut-off valve to close (RED) completely.</p> 	<p>② Set the scale of the flow regulator to "0".</p> 

※ Please place the cylinder under the seat in front .

If it does not fit in, please hold it on the lap during takeoff and landing.