



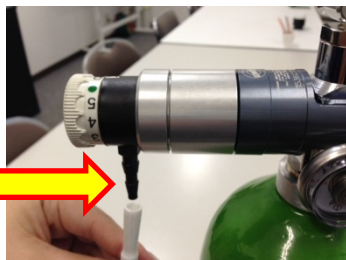



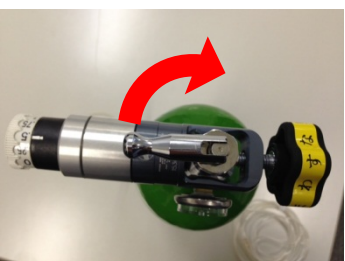



Thank you for boarding.

For your information, airport staff and cabin attendants cannot explain or operate an oxygen cylinder.

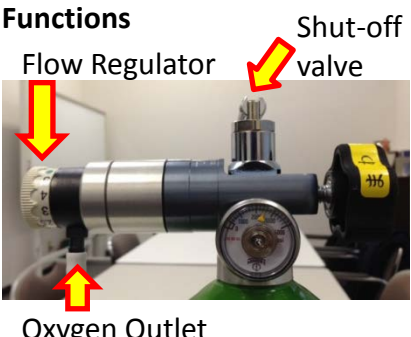

Instructions for Oxygen Cylinders 【International】

Functions 	Attachments Mask  Cannula 	Caution Don't touch! 
How to use <p>① Install a Mask or Cannula into the Oxygen Outlet.</p> <p>Caution Don't turn around Oxygen Outlet.</p> 	<p>② Check the dial of flow regulator is set to "0".</p> 	
<p>③ Turn the shut-off valve 1/4 turn CCW.</p> 	<p>④ Set the flow of oxygen according to your physician's instruction by turning the dial.</p> 	
Method of the end <p>① Turn the shut-off valve to close (CW).</p> 	<p>② Set the scale of the flow regulator to "0".</p> 	

✘ Please place the cylinder under the seat in front .

If it does not fit in, please hold it on the lap during takeoff and landing.